D/GB

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Translation of the original user manual

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**CE-Declaration of conformity**

The company AAT Alber Antriebstechnik GmbH herewith declares that the v-max is conform to the current regulations of the European Union, EEC Guidelines 2006/42/EG, 2007/47/EG and 93/42/EG. This declaration becomes void, if the v-max has been altered without the consent of AAT Alber Antriebstechnik GmbH.

Signature of the manufacturer:

Markus Alber  
(Managing shareholder)

The v-max has been successfully tested according to the standards DIN EN 12182, DIN EN 12184 and the applicable parts of ISO 7176 at the Technical University of Berlin, Berlin-Cert.

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## Table of content

<b>1</b>	<b>Symbols and safety instructions.</b>	<b>07</b>
<b>2</b>	<b>Description of the product</b>	<b>11</b>
2.1	Features of the device	11
2.2	Technical data	15
2.3	Dimensioned sketch	17
2.4	Scope of delivery	18
2.5	Accessories	19
<b>3</b>	<b>Assembly</b>	<b>20</b>
3.1	Wheels with quick-release axle	20
3.2	Battery pack	21
3.3	Power unit	22
3.4	Grips	23
3.5	Anti-tilt supports	24
3.6	Disassembly	25
<b>4</b>	<b>Initial operation</b>	<b>26</b>
4.1	Turn on the v-max	26
<b>5</b>	<b>Operation</b>	<b>27</b>
5.1	Driving the v-max.	29
5.2	Transport.	31
<b>6</b>	<b>Care and maintenance</b>	<b>32</b>
6.1	Batteries	32
6.2	Cleaning/Desinfection	39
6.3	Recycling.	40
6.4	Safety checks	41
6.5	New owner	41

<b>7</b>	<b>Warning notices, malfunction and trouble shooting</b>	<b>42</b>
7.1	Warning notices	.42
7.2	Trouble shooting	.44
<b>8</b>	<b>Warranty and liability.</b>	<b>45</b>
8.1	Warranty	.45
8.2	Liability.	.46

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## 1 Symbols and safety instructions

**WARNING** - Can lead to serious injuries or death!

**CAUTION** - Can lead to light injuries!

**NOTE** - Can lead to material damage!

- » Before initial operation, read the user manual and take note of the warning and safety instructions as well as the wheelchair's limiting values.
- » Before initial operation, check the electronic function, the anti-tilt supports, brackets and, if applicable, the steering rollers' air pressure.
- » Before initial operation, store the v-max at least one hour in a non-freezing environment.
- » Take note of the technical data.
- » The approved total weight may not be exceeded.
- » Due to the mounting of the v-max the adjustment of the seat/backrest can be restrained and the measurements of the wheelchair can change.
- » Operation only by one trained person. This person must be physically and mentally able to handle the v-max safely and must not suffer from cognitive impairment.
- » If possible, always operate with both hands.
- » Practice outside on level ground and at the lowest possible speed.

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- » The wheelchair's locking brakes must be adjusted properly.
- » Turn off the v-max for transportation.
- » During transportation in vehicles, secure the v-max according to the relevant legal regulations.
- » Do not use the v-max plus wheelchair as a seat in a vehicle.
- » Charge the battery pack after each use.
- » Never carry the charger at the charging cable.
- » When charging the v-max, make sure the cable lies on safe ground.
- » Never short-circuit or connect the battery pack's poles.
- » Keep the plug-in contacts dry.
- » Protect the v-max against humidity.
- » Protect the charger and the voltage converter for the car from oils, grease, aggressive cleaning agents, thinners or damage.
- » Do not use a high-pressure cleaning device.
- » Toothed segments and driveshafts must be cleaned of dirt and deposits.
- » Clean the wheelchair according to the manufacturer's specifications.
- » Never mend or bridge defect fuses.
- » Remove the charger/power cable before exchanging the fuses.
- » We recommend a safety check every two years.
- » Repairs and safety checks may only be done by the manufacturer or authorized personnel.
- » The surface temperature could increase due to the influence of external sources like sunlight.
- » Keep the v-max away from ignition sources.

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Power unit

Bracket for the power unit

Locking bolt for the anti-tilt supports

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## 2.2 Technical data

Technology	v-max
Measurements	Power unit: Height = 244 mm Width = min. 420 mm - max. 780 mm Depth = 128 mm
Weight	Power unit incl. grip and Cables = 8.2 kg Battery pack = 10.2 kg Power cable = 0.2 kg Battery bag plus velcro straps = 0.2 kg Battery charger = 0.3 kg Total weight (without wheelchair) = 19.1 kg
Total weight approved	200 kg <sup>(1)</sup> (person, wheelchair, v-max)
Limiting values for mounting	min. seat width 36, maximum wheel camber 5%
Speed, forward/backward	6 km/h   3 km/h
Range with one charge of the batteries	up to 19 km <sup>(2)</sup>
Braking - / climbing capacity	up to 20% <sup>(3)</sup>
Batteries	2 x 12 V/16 Ah
Direct current motor	24 V / 130 W
Noise level	The device's A weighted noise level lies typically below 66 dB (A)

Keys	Rocker switch, forward; 3 N Rocker switch, backward: 4 N Panic switch, forward: 40 N Panic switch, backward: 30 N Keys: On/Off, horn, speed pre-selection): 0,74 N
Storing/operating temperatur: device	-20° C up to +60° C / -10° C up to +50° C
Storing/operating temperatur: battery pack	-20° C up to +60° C / -10° C up to +50° C
Vibrations	The weighted RMS value of acceleration the upper body is exposed to lies typically below 2,5 m/s <sup>2</sup> .
Class of application	A, B
Sticker serial number	Power unit

<sup>(1)</sup> Please observe the maximum load according to the wheelchair manufacturer's stipulations.

<sup>(2)</sup> According to ISO 7176-4, the range depends on the person's weight, the underground, the territory, the temperature, etc.

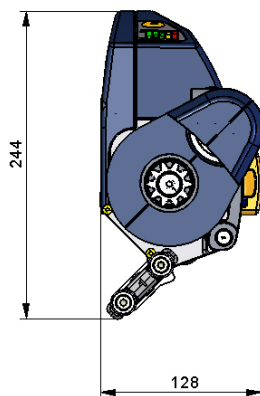
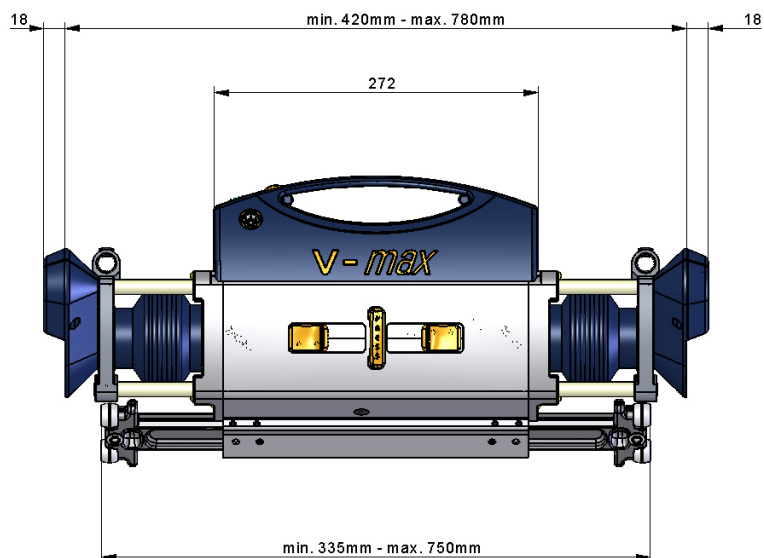
<sup>(3)</sup> At an approved weight of 130 kg, the braking - / climbing capacity also depends on the underground, the territory, the temperature, etc.

In the interest of progress we reserve the right for technical changes.

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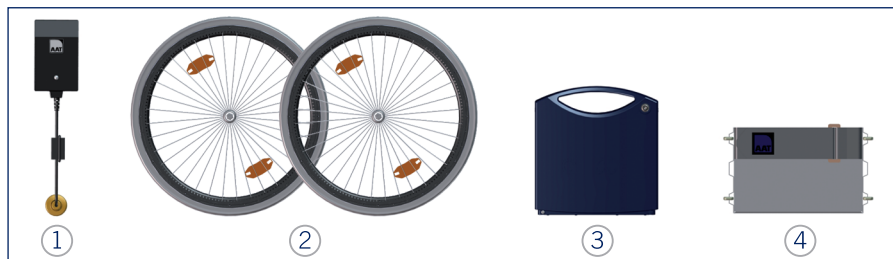
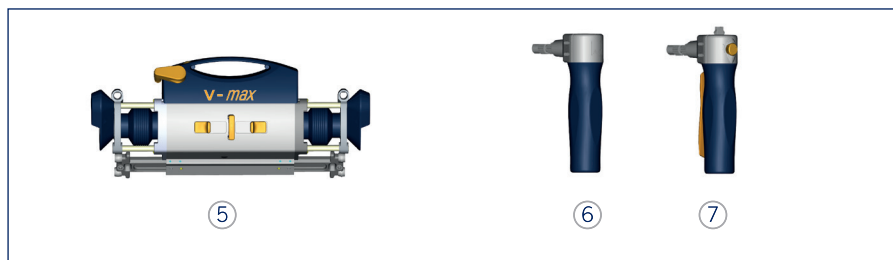
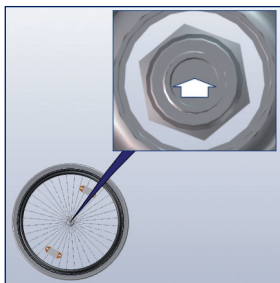
WARNING - The wheels with quick-release axles must lock!

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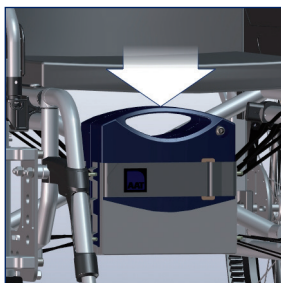
Insert into the bag and close the Velcro fastening.

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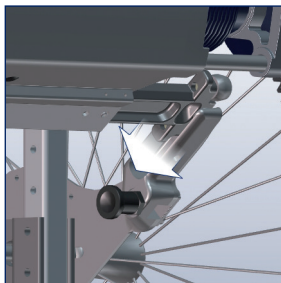
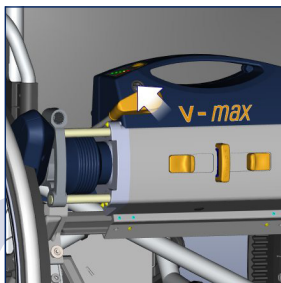
WARNING - Always tighten tightening knobs!

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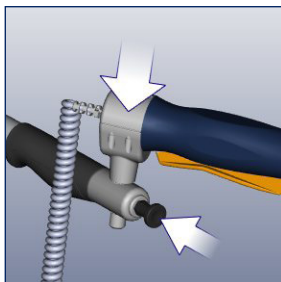
Put the grip into the back grip adaptor and make sure the locking bolt catches.

- » Control grip on the right!
- » Pushing grip on the left!

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**WARNING** - Never use the v-max without both anti-tilt supports!

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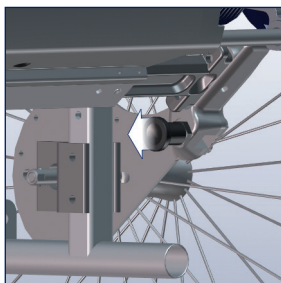
**WARNING** - Before initial operation check the anti-tilt supports!

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Should there be no anti-tilt support on your wheelchair – use AAT anti-tilt support in pairs!

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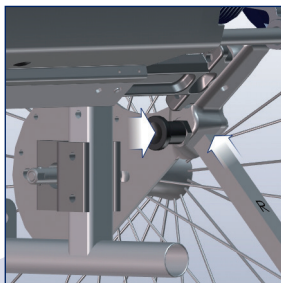
Slide into the bracket and make sure the locking bolt catches.

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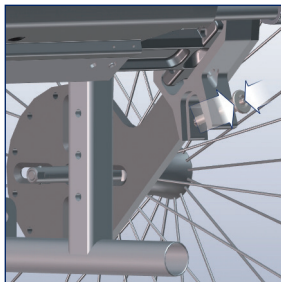
**Bracket with Quickpin**

Press the button and pull  
out the quickpin.

Slide into the bracket and  
secure it with the quickpin.

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**WARNING** - The wheelchair's locking brakes need to be adjusted properly!

**WARNING** - Before initial operation, check electronic function, anti-tilt supports and brackets!

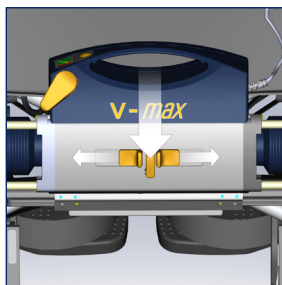
**NOTE** - Before initial operation, store the v-max at least one hour in a non-freezing environment!

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
## 4.1 Turn on the v-max

Use the PRESS button to move out the drive shafts.

- » Both sliders release.
- » The drive shafts extend.

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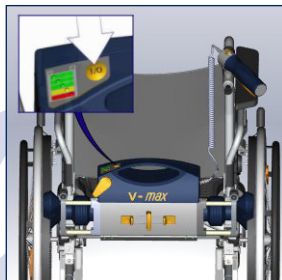WARNING - Operation only by one trained person! This person should be physically and intellectually capable of handling the v-max safely!

WARNING - Practice outside on level ground and at lowest speed possible!

WARNING - Never use the v-max without anti-tilt supports!

WARNING - Use the lowest possible speed when driving on slopes!

WARNING - Always go backwards over curbs/obstacles!

WARNING - Do not drive parallel to steep mountain sites or tracks!

WARNING - Keep a safe distance to curbs!

WARNING - Do not drive too close to stairs or precipices!

WARNING - When crossing streets, intersections and train tracks, use extra caution!

WARNING - Avoid using the v-max in heavy rain, ice, snow and on unsecured grounds (e.g. sand, mud, loose gravel)!

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WARNING - Avoid driving down slopes manually!

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WARNING - When not in use, e.g. while using mass transportation (bus, subway, etc.) or when you change seats, turn off the v-max, unplug the power cable and engage the locking brakes on the wheelchair!

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WARNING - Do not use the v-max on escalators, stairs or moving belts!

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WARNING - Driving on public roads, observe traffic regulations!

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WARNING - Always operate with both hands!

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CAUTION - Never use grips to lift or push, instead use the wheelchair's pushing grips!

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NOTE - Do not drive backwards to a wall with mounted anti-tilt supports!

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**WARNING** - Never touch rotating or moving parts!

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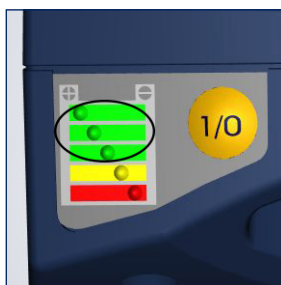
**CAUTION** - If the drive elements block or the driving characteristics change, turn off the v-max immediately and stop using it!

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Note the chapter on warnings, error indications and trouble shooting!

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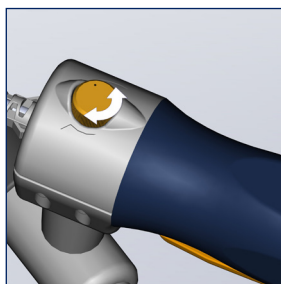
Red = empty

- » Charge battery!
- » Audio warning every 12 sec.!
- » The capacity of the batteries is sufficient for max. 20 - 35 steps!

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
Choose driving speed.

- » Practice at the lowest possible speed.

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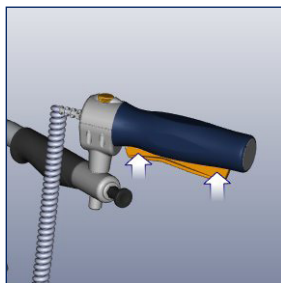
Press rocker switch in the direction you want to go.

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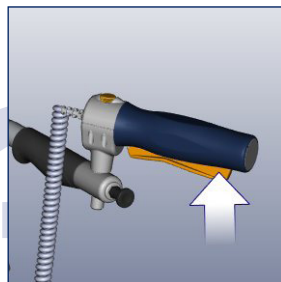### Panic switch

Press panic switch all the way

- » v-max is free running.
- » v-max brakes when releasing the rocker switch.
- » Independent of travel direction!

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**WARNING** - Do not use the v-max plus wheelchair as a seat in a vehicle!

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**WARNING** - During transportation in vehicles, secure the v-max according to relevant legal regulations!

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**CAUTION** - Turn off the v-max for transportation!

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**CAUTION** - Observe weight specifications!

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**NOTE** - Do not lay down the wheels with quick-release axle on their handrims!

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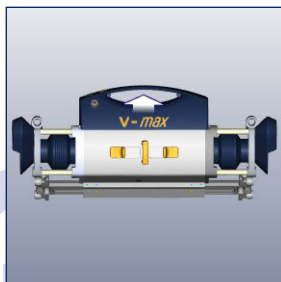
The battery pack is approved for air freight by DOT and IATA. However, inform airline ahead of time.

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WARNING - We recommend a safety check every two years! Inspection plate informs about the latest date for next inspection!  
Tip: We recommend a safety check, if the v-max goes to a new user!

WARNING - Repairs and safety checks may only be done by the manufacturer or authorized personnel!

### 6.1 Batteries

WARNING - Never short-circuit or connect the battery pack's poles!

WARNING - Remove the charger/ power cable before exchanging the fuses!

CAUTION - When charging the v-max, make sure the cable lies on safe ground!

CAUTION - Never mend or bridge defect fuses!

NOTE - Protect the v-max against humidity!

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**NOTE** - Never carry the charger at the charging cable!

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**NOTE** - Charge the battery pack after each use!

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**NOTE** - Protect the charger and the voltage converter for the car from oils, grease, aggressive cleaning agents, thinners or damage!

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**NOTE** - Observe the specifications on the charger!

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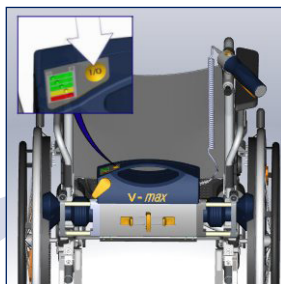
Battery pack is admitted to air travel by DOT and IATA (observe chapter 5.5).

The battery-pack is maintenance free and rechargeable.

It is not possible to use the v-max while it is charging.

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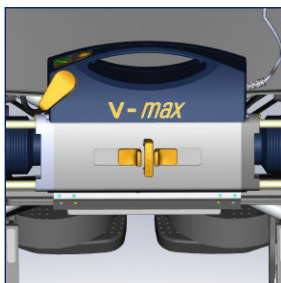
If desired:  
Put the grips back into the adaptors on the device.

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Move the two sliders in direction to the PRESS button.  
» Both sliders engage.

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Unplug the power cable.

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Connect charger.

D

GB

LED grün = Akku voll  
» ca. 10 Stunden

LED green = batteries full  
» approx. 10 h

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
LED orange = batteries are  
being charged

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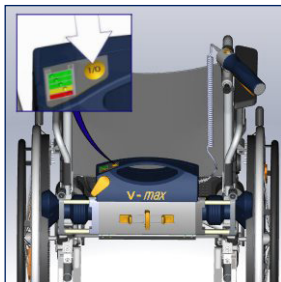
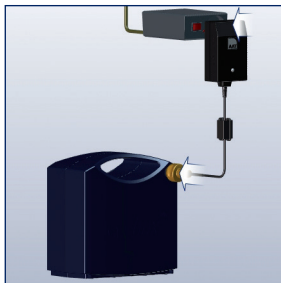
Unfasten Velcro fastener and remove battery pack.

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Connect the voltage  
converter for the car and  
the charger.

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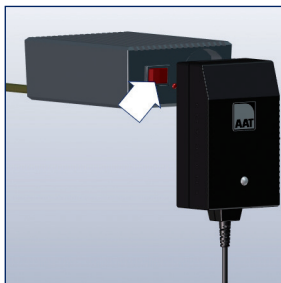Turn on voltage converter  
for the car.  
» Acoustic signal = low  
voltage!

LED grün = Akku voll  
» ca. 10 Stunden

LED green = batteries full  
» approx. 10 h

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
LED orange = batteries are  
being charged

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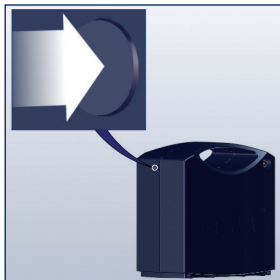
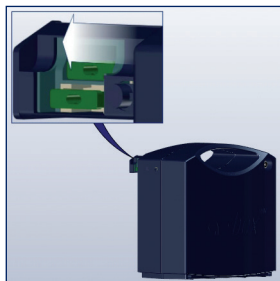
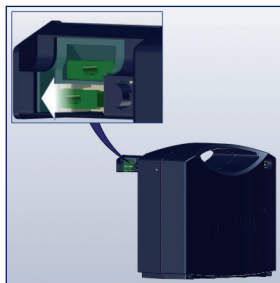
Change fuse.

» Upper fuse = spare fuse!  
» Spare fuses available at  
the manufacturer.

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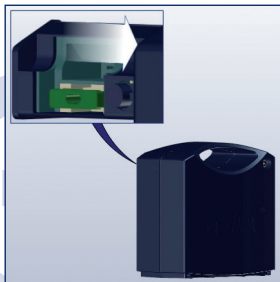Push in fuse box.

» Make sure the locking  
mechanism catches.

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NOTE - Keep the v-max away from ignition sources!

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NOTE - Toothed segments and driveshafts must be cleaned of dirt and deposits.

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NOTE - Clean the wheelchair according to the manufacturer's specifications!

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NOTE - Do not use a high-pressure cleaning device!

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With a moist cloth and mild household cleaners / disinfectants.

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### 6.3 Recycling

The v-max and its battery pack are products of long durability.

However, if your v-max is old and has reached the end of its lifespan you may return its components to AAT Alber Antriebstechnik GmbH or authorized personnel to be recycled.

A new law concerning electric and electronic devices (ElektroG) has been passed on March 24, 2005 regulating the taking back and recycling of old electric and electronic devices.

According to the Federal Ministry for the Environment (German BMU) the v-max is considered a means of transportation and does not fall under this law concerning the recycling of old electric or electronic devices (ElektroG, §2 paragraph 1 sentence 1 (cross reference, Guideline of the European Parliament 2002/96/EG)).

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**WARNING** - Safety checks may only be done by the manufacturer or authorized personnel!

**WARNING** - We recommend a safety check every two years! Inspection plate informs about the latest date for next inspection!  
Tip: We recommend a safety check, if the v-max goes to a new user!

A safety check is mandatory before the device changes hands. To conduct a safety check it is necessary to attend a training course. Please check the homepage for further information

## 6.5 New owner

The v-max is a device suitable for multiple use. Before the v-max changes hands it must be cleaned, disinfected and pass a safety check (see chapter 6.2).

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#### Malfunction

Red LED blinks	Type of Malfunction
1x	Malfunction of the motor, the motor controller or the battery pack
2x	Low voltage, batteries empty
3x	Overvoltage
4x	Electro-magnetic brake at fault
5x	Indicator for excess voltage
6x	Indicator for oversteering
7x	Temperature controller

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## 7.2 Trouble shooting

Problem	Check and potential solution
v-max cannot be turned on	Is the power cable plugged in the power unit and in the battery pack? No? Plug it in.
	Is the battery pack empty? Yes? Charge it.
	Is the fuse defective? Yes? Change it.
	Problem still there? Yes? Contact the manufacturer or authorized personnel!
v-max does not drive	Is the v-max still turned on? No? Turn it on.
	Immobiliser activated? Yes? deactivate!
	Problem still there? Yes? Contact the manufacturer or authorized personnel!
Battery pack doesn't charge	Is the charger connected? No? Connect it properly.
	Problem still there? Yes? Contact the manufacturer or authorized personnel!
Fuse often damaged?	Yes? Contact the manufacturer or authorized personnel!

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## 8 Warranty and liability

### 8.1 Warranty

AAT Alber Antriebstechnik GmbH grants a warranty of two years for the v-max including all its accessory parts (with the exception of the battery pack). The warranty begins at the date of purchase and covers failure of material and errors in manufacturing.

For the battery pack, always assuming correct maintenance AAT Alber Antriebstechnik GmbH grants a warranty of six months from the date of purchase.

The warranty becomes void concerning defects or damage which are the result of the following:

- » damage due to excessive strain.
- » damage due to improper handling.
- » violent damage.
- » natural wear and tear (driving belt, toothed segments, wheelchair wheels, driveshafts, keypad).
- » improper and insufficient maintenance.
- » unauthorized constructional changes or inappropriate use of the v-max or its accessories.
- » Using the v-max exceeding the approved total weight.
- » improper charging of the battery pack.

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## 8.2 Liability

The manufacturer AAT Alber Antriebstechnik GmbH, is neither liable nor responsible for potential damage of the v-max in case of:

- » improper handling of the v-max.
- » repairs, fittings or other works performed on the v-max by unauthorized personnel.
- » operation not in accordance with the user manual.
- » extraneous parts added or connected to the v-max.
- » parts removed from the v-max.
- » operation exceeding the approved total weight.
- » no regular biannual maintenance check performed by AAT Alber Antriebstechnik GmbH or its authorized personnel.
- » operation of the v-max with insufficiently charged batteries.

Important EMI-Information:

- » 20 Volts per meter (V/m) is a generally achievable and useful immunity level against interference from radio wave sources (as of May 1994) (the higher the level, the greater the protection).
- » This device has an immunity level of 20 V/m with no accessories connected to the device.
- » Nevertheless, always check whether the device's electronic functions work properly before each use. This is particularly important, if there is an aerial mast nearby.
- » Please note that the v-max may interfere with other electronic devices' sensitive electromagnetic fields (e.g. anti-theft systems in department stores). This may also happen, if you connect other devices.
- » Also make sure that the v-max is not operated in close proximity to medical devices with a high potential for danger and/or life-supporting functions as well as diagnostic devices.

D

GB

Notizen:

Notes:

48

D

GB

Notes:

Notizen:

49

D

GB

Notizen:

Notes:

50

D

GB

Notizen:

Notes:

51

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